


Manufacturers use different configurations and types of fixings to mount the upholstery sections to their couch frames, replacing upholstery can be very difficult, if you do not feel competent to carry out this work, then please do not proceed. Trimbio can offer you a fitting service, subject to location, please contact us for a quotation.

The two most common ways to fix upholstery to couch frames are self tapping screws or lugs and bolts. Self tappers just go through the frame and screw into the upholstery section. Lugs are required to be mounted into the upholstery section by drilling a hole for them to screw into, then the bolt goes through the frame and screws into the lug.


### Self Tapper Option

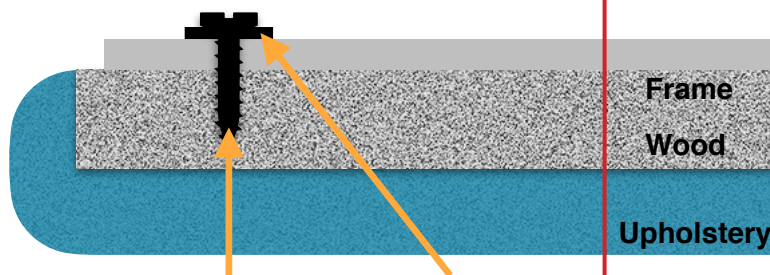

Self tapper

Washer

### Lug & Bolt Option

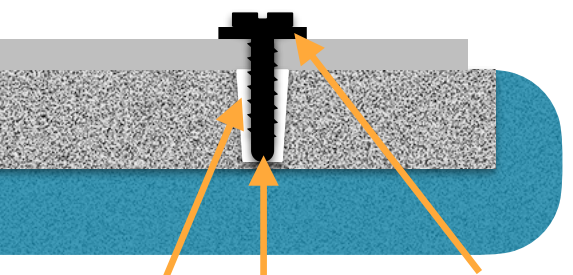

Lug

Bolt

Washer

We are happy to supply you with either, for you to fit. ( Please note that once used / fitted it may not be possible to move / reposition these screws / lugs and new upholstery may be required to overcome any mistakes or errors made).

For each section of the couch we require the depth of the frame ( measure as shown below) and the quantity of screws for that thickness of frame. Please supply this information in the upholstery order form.

